

Cord Hooks

Quick Installation and Maintenance Instructions



CRDHLDLDR/CRDHLDLDR2

Medical Facility Responsibility

Preventive maintenance checks must be performed regularly to maintain the quality and performance of this product. Any parts that may be broken, missing, worn, distorted or contaminated in any way should not be used and all affected parts should be replaced immediately. Should the necessity of any repair be suspected; please contact Amico Accessories Inc. or your local distributor.

Installation Tools



#2 Phillips Head Screwdriver




3/8" (9.5 mm) Wrench




NOTE: Amico Accessories Inc. does not provide any tools or hardware necessary for the installation.

Installation Reference

CRDHLDLDR

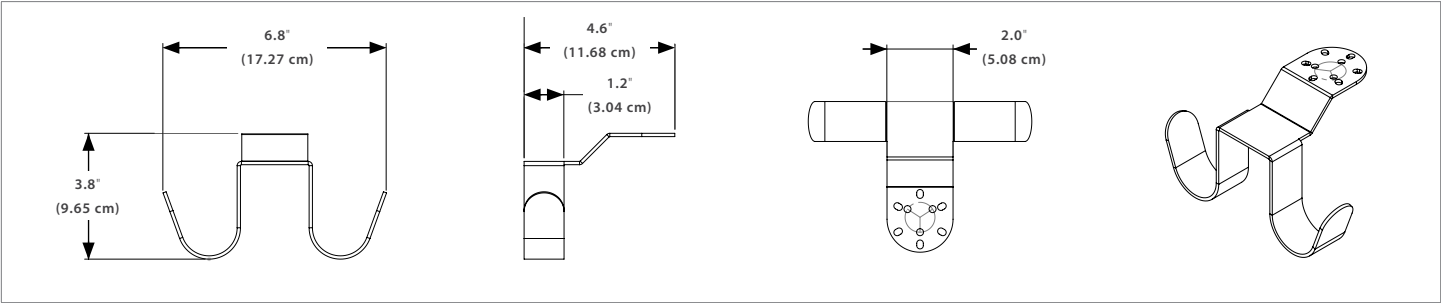

CRDHLDLDR2

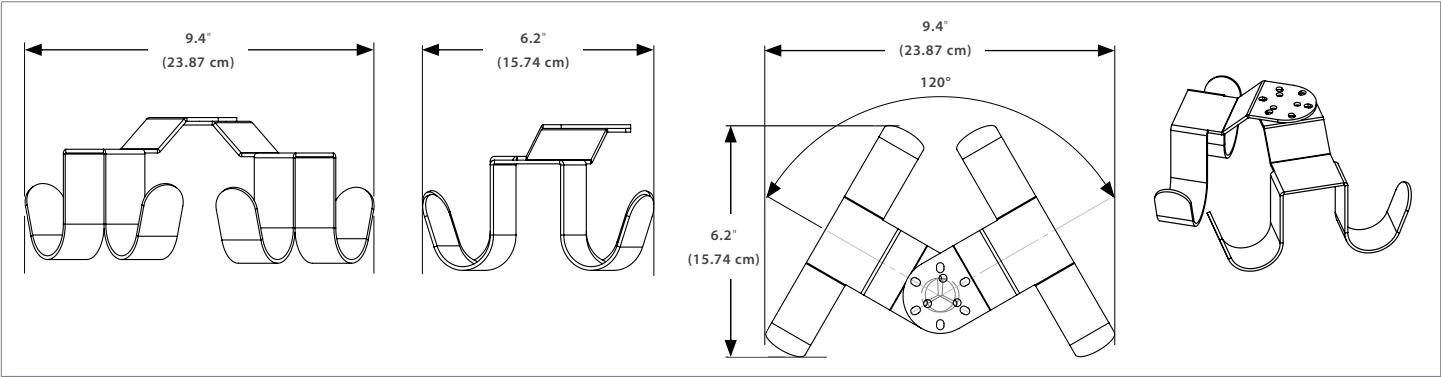

Maintenance/Cleaning



WARNING: CRDHLDLDR/CRDHLDLDR2 require periodic inspection and maintenance to perform optimally and achieve maximum operation life. The intervals shown are recommended. Maintenance schedules should be more frequent for higher use areas.

Area	Maintenance	Period
Around screws	Ensure screws are fastened securely, with no gaps.	6 months

Warranty

Our standard warranty is available for download at: www.amico.com/warranties.

Cord Hooks

Quick Installation and Maintenance Instructions




CRDHLDLDR/CRDHLDLDR2

CRDHLDLDR

For use with CRDHLDLDR at the bottom of DOWNPOST-()


Install 3x H-MPF-1032-12 through 3x H-WLM-10, CRDHLDLDR, MON-CPU-DISK, and into 3x H-ANUT-1032.

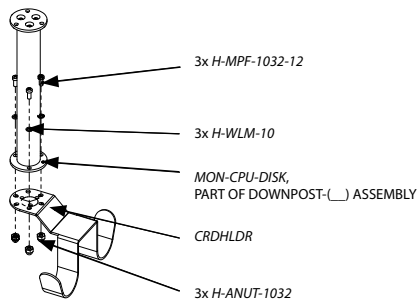

CRDHLDLDR2


For use with CRDHLDLDR2 at the bottom of DOWNPOST-()

Install 3x H-MPF-1032-58 through 3x H-WLM-10, 2x CRDHLDLDR, MON-CPU-DISK, and into 3x H-ANUT-1032.

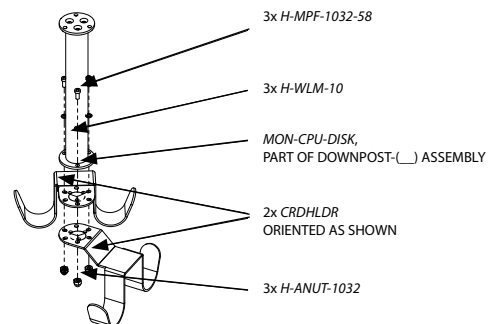


Sandwiched Between DOWNPOST-() and AHM/SSM/EXT

Install 3x H-MPF-1032-34 through 3x H-WLM-10, CORDWRAP, MON-CPU-DISK, and into MON-SWIV-DISK.

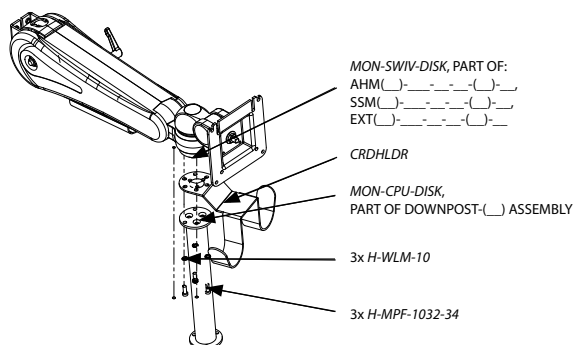


Sandwiched Between DOWNPOST-() and AHM/SSM/EXT

Install 3x H-MPF-1032-34 through 3x H-WLM-10, 2x CRDHLDLDR, MON-CPU-DISK, and into MON-SWIV-DISK.

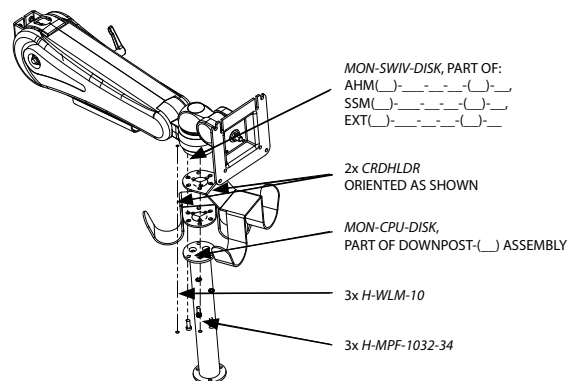

Directly to AHM/SSM/EXT

Install 3x H-MPF-1032-12 through 3x H-WLM-10, CRDHLDLDR, and into MON-SWIV-DISK.

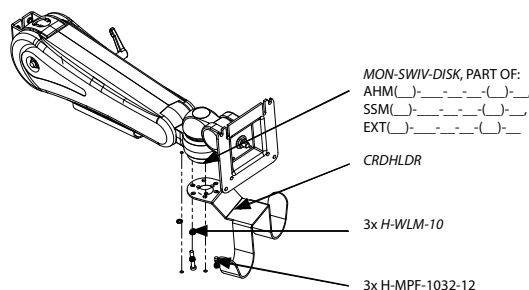

Directly to AHM/SSM/EXT

Install 3x H-MPF-1032-34 through 3x H-WLM-10, 2x CRDHLDLDR, and into MON-SWIV-DISK.

