

Counterweight Plate for Hummingbird Cart

Quick Installation and Maintenance Instructions



HMA-VS-CW-PLT-00

Medical Facility Responsibility

Preventive maintenance checks must be performed regularly to maintain the quality and performance of this product. Any parts that may be broken, missing, worn, distorted, or contaminated in any way should not be used and all affected parts should be replaced immediately. Should the necessity of any repair be suspected; please contact your local distributor.

⚠ WARNING: It is the responsibility of the end user to ensure all aspects of installation are covered by following the Workstations manual and reviewing warnings/notes found at:

<https://docs.amico.com/amico-aa-mobile-computer-workstation-hummingbird-lite-manual.pdf>
<https://docs.amico.com/amico-aa-mobile-computer-workstation-hummingbird-manual.pdf>

Installation Tools




#2 Phillips Head Screwdriver




Flat Head Screwdriver


Installation Reference

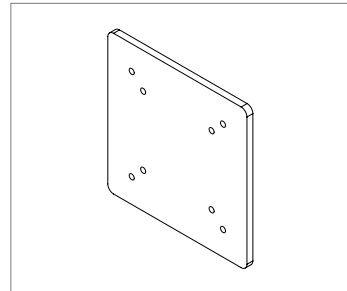

Counterweight Plate
(X-AHM-FITW-CW-SS)

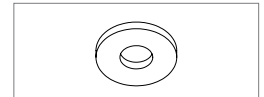

8x Spacers
(H-WASH-M4-ST)

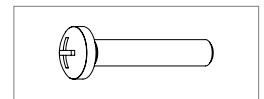

4x M4 22 mm Screws
(H-PHMS-M4-22)

Installation Instruction for Hummingbird and Hummingbird Lite Workstation

i NOTE: COUNTERWEIGHT (AHM-FITW-CW-PLT-00) SHOULD ONLY BE INSTALLED AFTER SECURING THE CART BY LOCKING THE CASTERS. IT IS THE RESPONSIBILITY OF THE CUSTOMER TO VERIFY THAT THE CART HAS BEEN INSTALLED IN ACCORDANCE WITH AMICO INSTALLATION INSTRUCTIONS.

1. Lock the **CASTERS** by stepping on the **LOCKING LEVER** (on each caster) such that it clicks in the down position (**Figure 1**) and move to instruction step 2-6.

⚠ WARNING: KEEP THE HUMMINGBIRD CART AT LOWEST POSITION AND DISCONNECTED FROM POWER OUTLET.

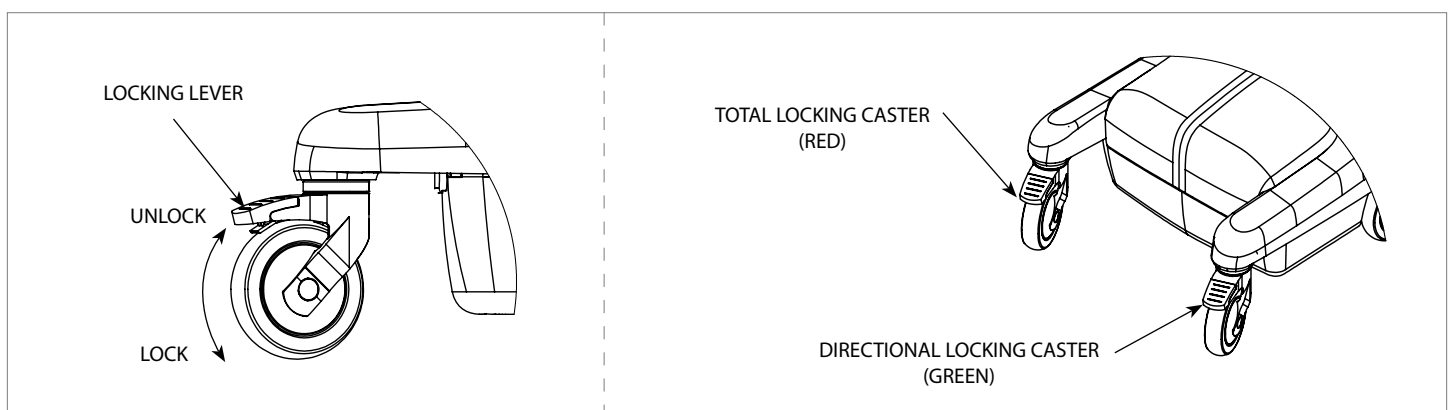


Figure 1

Counterweight Plate for Hummingbird Cart

Quick Installation and Maintenance Instructions



i NOTE: IF THE *DISPLAY* IS ALREADY INSTALLED ON THE WORKSTATION, REMOVE BOTTOM **2X M4 SCREWS** AND LOOSEN TOP **2X M4 SCREWS**. CAREFULLY LIFT THE *DISPLAY* OUT OF THE *VESA HEAD* THEN REMOVE TOP **2X M4 SCREWS** OTHERWISE PROCEED WITH NEXT STEP.

2. Determine the *DISPLAY* mounting configuration. VESA 75 (75 mm x 75 mm M4 thread) or VESA 100 (100 mm x 100 mm M4 threads).
3. Partially thread **2X M4 22 MM SCREWS** (provided) through the *COUNTERWEIGHT PLATE* and into top 2x VESA holes on the *DISPLAY* (not provided) (**Figure 2**).
4. Hang the *DISPLAY* and *COUNTERWEIGHT PLATE* from 2x slots on the VESA Head (for VESA 100 use outer slots, for VESA 75 use inner slots. Ensure slot opening is facing up for the configuration needed (**Figure 2**).
5. Insert remaining **2X M4 22 MM SCREWS** through the *COUNTERWEIGHT PLATE* and into the *DISPLAY*. Tighten all **SCREWS** (**Figure 2**). If the thread depth is short in the *DISPLAY*, place 1x or 2x *SPACERS* between the VESA Head and *SCREW* as required.

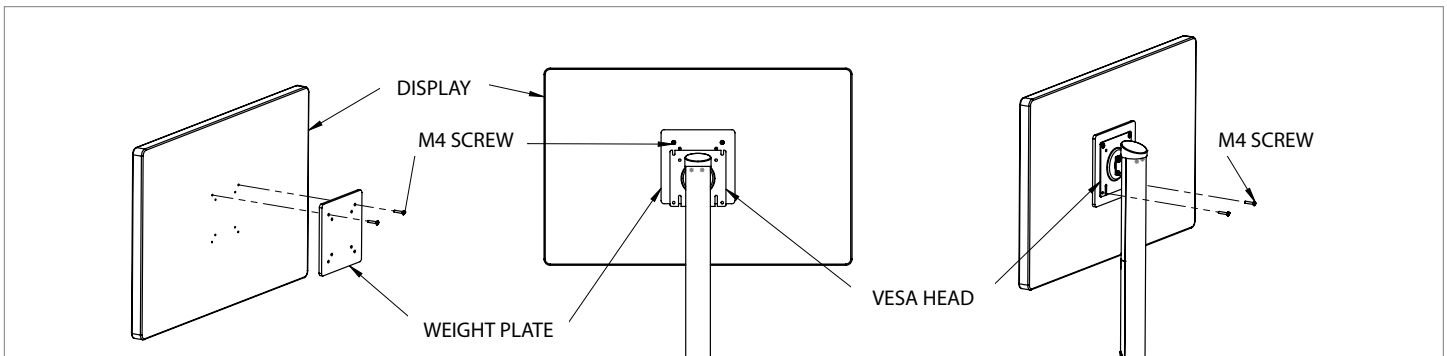


Figure 2

Troubleshooting

Symptom	Possible Cause	Solution
Display is wobbling.	Screw length is more than display depth, use the spacers between screw and VESA Head or screws engaged fully.	Tighten the screws.

Refer to the Workstation manuals for additional troubleshooting instructions:

<https://docs.amico.com/amico-aa-mobile-computer-workstation-hummingbird-lite-manual.pdf>

<https://docs.amico.com/amico-aa-mobile-computer-workstation-hummingbird-manual.pdf>

Maintenance

It is recommended that once every three (3) months, all bolts and screws are checked to ensure that they are tight. Visually inspect the product for signs of damage based on degree of usage.

Counterweight Plate for Hummingbird Cart

Quick Installation and Maintenance Instructions



Warranty

Amico Accessories Inc. warrants all mounting accessories to be free from defects in material and workmanship for a period of twelve (12) months from the date of shipment. Within this period Amico Accessories Inc. will repair or replace any part which is proven to be defective.

Amico Accessories Inc. will warrant its materials to be free from defect for an additional period of four (4) years, (five [5] years from the date of shipment). Within this period, Amico Accessories Inc. will replace any part which is proven to be defective, at no charge. Shipping and Installation costs after the first twelve (12) months will be borne by the Customer.

This warranty is valid only when the product has been properly installed according to Amico Accessories Inc. specifications, used in a normal manner and serviced according to factory recommendations. It does not cover products that are not manufactured by Amico Accessories Inc. It does not cover failures due to damage which occurs in shipments or failures which result from accidents, misuse, abuse, neglect, mishandling, alteration, misapplication or damage that may be attributable to acts of God.

Amico Accessories Inc. shall not be liable for incidental or consequential damages resulting from use of equipment.

All claims for warranty must first be approved by Amico Accessories Inc. A valid Return Goods Authorization (RGA) number must be obtained from Amico Accessories Inc. prior to commencement of any service work. Warranty work, which has not been pre-authorized by Amico Accessories Inc., will not be reimbursed.

AMICO ACCESSORIES INC. DOES NOT HONOR VERBAL STATEMENTS CONCERNING THE WARRANTY.

The distributor and/or dealer are not sanctioned to create verbal warranties about the product described in this agreement. Any statements will not be honored or be made part of the agreement of sale. This document is the final complete and exclusive terms of the agreement.

THIS WARRANTY IS INCLUSIVE AND REPLACES ALL OTHER WARRANTIES.

Amico Accessories Inc. shall not, under any circumstances be liable for incidental or consequential damages including, but not limited to, profit, loss of sales or injuries to person(s) or property. Correction of noncompliance as noted above will result in completion of all liabilities of Amico Accessories Inc., whether based on agreement, neglect or changed materials, designs or specifications without notice.

All claims for warranty must first be approved by Amico Accessories Inc. Service Department: accessories@amico.com or 1.877.264.2697. A valid Return Goods Authorization number must be obtained from Amico Accessories Inc. prior to commencement of any warranty claim.