

Cup Dispenser and Water Bottle Holder

Quick Installation and Maintenance Instructions



Medical Facility Responsibility

Preventive maintenance checks must be performed regularly to maintain the quality and performance of this product. Any parts that may be broken, missing, worn, distorted, or contaminated in any way should not be used and all affected parts should be replaced immediately. Should the necessity of any repair be suspected; please contact your local distributor.



WARNING: It is the responsibility of the end user to ensure all aspects of installation are covered by following the full manual and reviewing warnings/notes found at <https://amico-apps.com/amico-website/files/downloads/amico-aa-mobile-computer-workstation-hummingbird-manual.pdf>

Installation Tool



7/32 Allen key
(NOT PROVIDED)

Installation Reference



Cup Dispenser



Water Bottle Holder



FTA Adapter

Installation Instructions for Water Bottle Holder

(HMA-WBHLD-F)



CAUTION: Avoid dropping/free falling the water bottle into the holder.



CAUTION: Secure the water bottle cap when it's unused to avoid spilling.



CAUTION: Secure the FTA adapter by tightening the 2x bottom set screws using a 7/32 Allen key. (Figure 1)

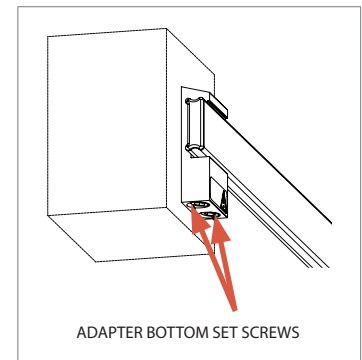


Figure 1

Specification

Dimensions (W x D x H): 3.5" x 5.0" x 10.4" (8.9 cm x 12.7 cm x 26.4 cm) **Weight:** 0.7 lbs (0.3 kg).

This product is designed to fit a water bottle which is less than or equal to 1 liter in volume, maximum diameter of 3.5" (8.8 cm), and weights no more than 2.5 lbs (1.13 kg).

1. Place the WATER BOTTLE HOLDER on the PERCH BAR (Figure 2) and secure by tightening the adapter bottom screw with a 7/32 Allen key. (Figure 1)

Note: Refer to the "Fairfield Adapter Installation" to install the holder. <https://docs.amico.com/amico-aa-fairfield-horizontal-rail-adapter-manual.pdf>

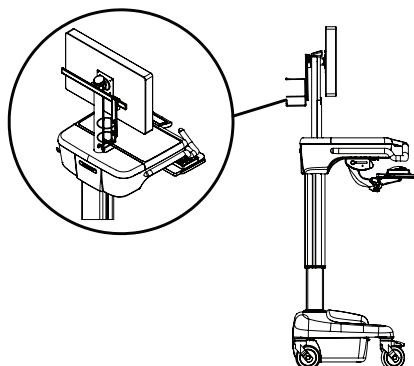


Figure 2

2. Place the water bottle inside the WATER BOTTLE HOLDER gently. (Figure 3)



Figure 3

Cup Dispenser and Water Bottle Holder

Quick Installation and Maintenance Instructions



Installation Instructions for Cup Dispenser

(HMA-DSP-MCUP-F / HMA-DSP-WCUP-F / HMA-DSP-MCUP-FB)

CAUTION: Secure the FTA adapter by tightening the 2x bottom set screws using a 3/32 Allen key. **(Figure 4)**

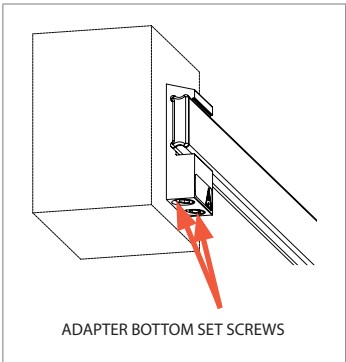


Figure 4

Specification

Purpose	Part Number	Dimensions (W x D x H)/Physical Weight	(For Reference Only)		
			Cup Type	Cups Size	Dispenser Capacity (Approx.)
Medication	HMA-DSP-MCUP-F	3.9" x 4.7" x 9.6" (9.9 cm x 11.8 cm x 24.3 cm) 1.0 lbs. (0.5 kg)	Paper	1 oz	90 pcs.
			Plastic	1 oz	120 pcs.
	HMA-DSP-MCUP-FB	3.9" x 5.5" x 12.3" (9.9 cm x 13.97 cm x 31.24 cm) 1.0 lbs. (0.5 kg)	Paper	1 oz	90 pcs.
			Plastic	1 oz	120 pcs.
Drinking Water	HMA-DSP-WCUP-F	3.9" x 4.7" x 9.6" (9.9 cm x 11.8 cm x 24.3 cm) 0.7 lbs. (0.3 kg)	Paper Cone	4 oz	115 pcs.
			Paper	4 oz	35 pcs.
				5 oz	30 pcs.
				6 oz	25 pcs.



Figure 5

Cup Dispenser and Water Bottle Holder

Quick Installation and Maintenance Instructions



Installation Instructions for Cup Dispenser on Angled Bracket (HMA-DSP-MCUP-FB)

1. Place the FTA ADAPTER on the PERCH RAIL (**Figure 6**) with 0.94" (2.4 cm) away from the POST bracket and secure by tighten the adapter bottom screw with $\frac{7}{32}$ Allen key. (**Figure 4 above**)

Note: Refer the "Fairfield Adapter Installation" to install the dispenser.



Figure 6

- Note:** Recommended place to avoid cup dispenser wobbling/vibrating while in transportation. Keep the FTA ADAPTER 0.94" (2.4 cm) away from the PERCH RAIL BRACKET. (**Figure 7**)

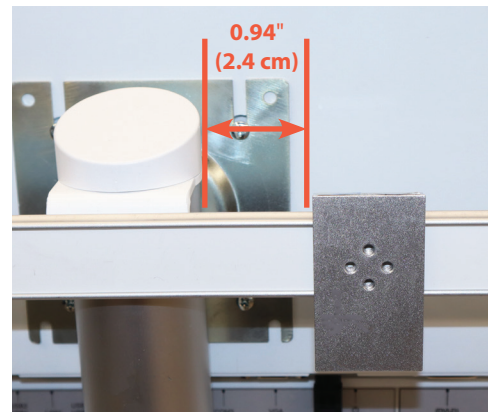


Figure 7

3. Align the CUP DISPENSER with mounting bracket on FTA ADAPTER & secure with 2x H-FHUCS-832-516 screws. (**Figure 8**)



Figure 8

4. Cover the holes with Amico label. (**Figure 9**)



Figure 9

Cup Dispenser and Water Bottle Holder Quick Installation and Maintenance Instructions

Installation Instructions for Cup Dispenser

(HMA-DSP-MCUP-F / HMA-DSP-WCUP-F)

1. Place the CUP DISPENSER (Medical/Water) on the PERCH BAR (**Figure 6**) and secure by tightening the adapter bottom screw (**Figure 4**) with a $\frac{7}{32}$ Allen key.

Note: Refer to the "Fairfield Adapter Installation" to install the dispenser.
<https://docs.amico.com/amico-aa-fairfield-horizontal-rail-adapter-manual.pdf>

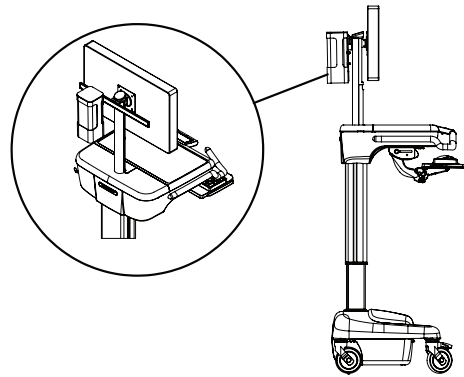


Figure 10

Instruction for Loading and Dispensing Cups

(HMA-DSP-MCUP-F, HMA-DSP-MCUP-FB, HMA-DSP-WCUP-F)

1. Remove the seal for the CUP DISPENSER first then open the lid. (**Figure 11**)



0-135° for HMA-DSP-MCUP-F / HMA-DSP-WCUP-F
0-90° for HMA-DSP-MCUP-FB



Figure 11

2. Load the medication cups for HMA-DSP-MCUP-F or water cups for HMA-DSP-WCUP-F into the dispenser up to the top then close the lid. (**Figures 12, 13**)



Figure 12

CAUTION: Don't load all the cups at once, split the cup count and load into the dispenser in portions.



Figure 13

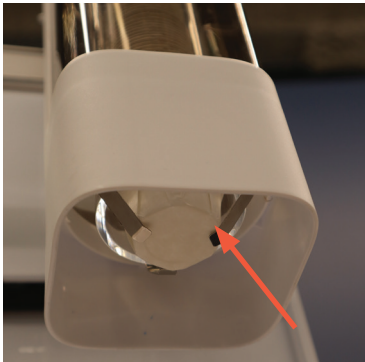
3. Rotate and pull out the cups. (**Figure 14**)

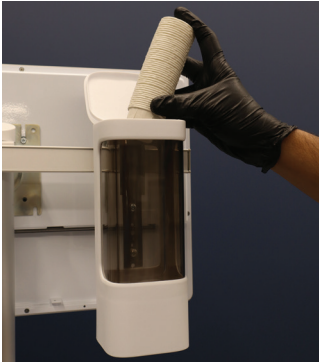


Figure 14

Troubleshooting

What it Looks Like	What Happened	How to Fix it
Bottle holder/dispenser glides on the perch bar.	Adapter locking set screws have loosened.	Tighten the adapter's locking set screws.
After loading, cups are not coming out of the dispenser.	The cups have not yet passed through the bottom of the dispenser springs. (Figure 15)	Load and push the cups down until the bottom cup comes out from the dispenser. (Figures 16,17)








Figure 15

Figure 16

Figure 17

Maintenance

It is recommended that once every three (3) months, all locks, bolts, and screws are visually inspected for signs of product wear and tear or damage. Check the product for losses in the mount or orientation, and ensure the adapter is secure.

Warranty

During the term of warranty: Within the first twelve (12) months from the date of shipment, Amico Accessories will repair or replace any part which is proven to be defective at no cost. After the twelve (12) month period until five [5] years from the date of shipment, Amico Accessories will send the parts to the customer free of charge, however shipping and installation will be borne by the customer.

The warranty is valid only when the product has been properly installed according to Amico Accessories specifications, used in a normal manner, and serviced according to factory recommendations. It does not cover failures due to damage which occur in shipments or failures which resulted from accidents, misuse, abuse, neglect, mishandling, alteration, misapplication or damage that may be attributable to Force Majeure.

AMICO ACCESSORIES DOES NOT HONOR VERBAL STATEMENTS CONCERNING THE WARRANTY.

The distributor and/or dealer are not sanctioned to create verbal warranties about the product described in this agreement. Any statements will not be honored or be made part of the agreement of sale. This document is the final complete and exclusive terms of the agreement.

THIS WARRANTY IS INCLUSIVE AND REPLACES ALL OTHER WARRANTIES.

Amico Accessories shall not, under any circumstances be liable for incidental or consequential damages including, but not limited to, profit, loss of sales or injuries to person(s) or property.

Correction of non-compliance as noted above will result in completion of all liabilities of Amico Accessories whether based on agreement, neglect or changed materials, designs or specifications without notice.

All claims for warranty must first be approved by Amico Accessories Customer Service Department: (info@amico-accessories.com or 1-877-264-2697). A valid Return Goods Authorization number must be obtained from Amico Accessories prior to commencement of any warranty claim.