# Mindray Satellite Module Rack (SMR) Mount **Quick Installation and Maintenance Instructions**



## **Medical Facility Responsibility**

Preventive maintenance checks must be performed regularly to maintain the quality and performance of this product. Any parts that may be broken, missing, worn, distorted, or contaminated in any way should not be used and all affected parts should be replaced immediately. Should the necessity of any repair be suspected; please contact your local distributor.

#### **Installation Tools**



1/4" Allen Key (NOT PROVIDED)

#2 Philips Screwdriver (NOT PROVIDED)

#### Installation Reference







SMR Adapter Plate



Hardware Kit



WARNING: It is the responsibility of the end user to ensure all aspects of installation are covered by following the full manual and reviewing warnings/notes found at https://docs.amico.com/amico-aa-ahm-adjustable-height-patient-monitor-arm-manual.pdf for Adjustable Height Monitor (AHM) Arm, or https://docs.amico. com/amico-aa-ssm-side-to-side-patient-monitor-arm-manual.pdf for Side to Side Monitor (SSM) Arm.


### Installation Instructions (MR-SMR-MNT-02)

## **Specifications**

Dimensions: (W x D x H): 5.1" x 2.5" x 1.6" (13.1 cm x 6.4 cm x 4.0 cm)

Weight: 1.1 lbs (0.5 kg) Load Rating: 15.0 lbs (6.8 kg)

# **Mounting to Device**

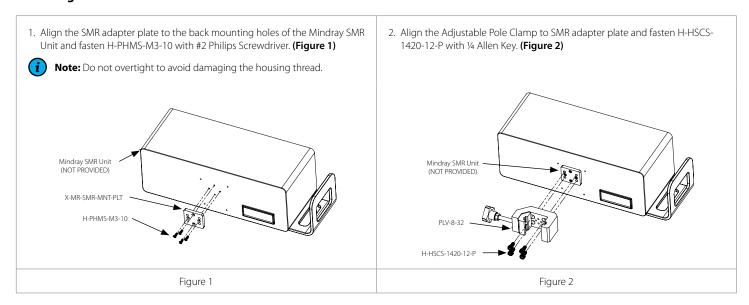

## **Mounting Device Assembly to Platform**

WARNING: DO NOT ATTEMPT TO MOUNT/REMOVE THE PLV-8-32 WHEN LOADED WITH ANY DEVICE.



WARNING: DO NOT MOUNT THE PLV-8-32 ABOVE THE PATIENTS HEAD.



WARNING: ENSURE POST IS STABLE AND PROPERLY INSTALLED BEFORE MOUNTING THE PLV-8-32.

# Mindray Satellite Module Rack (SMR) Mount **Quick Installation and Maintenance Instructions**



1. Ensure the handle HPC1-CLMP-HDL is loose (i.e. the handle is in its outer most position. If handle is not in its outer most position, then rotate the handle counter clockwise in order to make it so). Then, align the grooves of X-PC1-CLMP-CLP to the mounting surface at the desired height and rotate the handle (H-PC1-CLMP-HDL) clockwise until X-PC1-CLMP-CLP is completely tightened and secured on to the mounting platform. (Figure 3)



Note: For clarity, figures below show mounting of PLV-8-32 without any device attached.

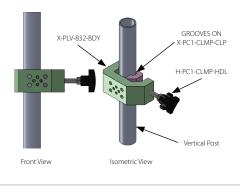


Figure 3

2. Tighten the nut (H-LNUT-M8-NYL) by rotating it clockwise onto the clamp body (X-PLV-832-BDY). (Figure 4)



WARNING: ENSURE THE X-PC1-CLMP-CLP IS COMPLETELY TIGHTENED AND SECURED ON TO THE MOUNTING PLATFORM BEFORE TIGHTENING THE NUT AND LOCKING THE PLV-8-32 ONTO THE MOUNTING PLATFORM.



Note: For clarity, figures below show mounting of PLV-8-32 without any device attached.

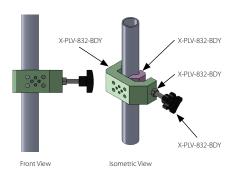


Figure 4



Note: To uninstall, rotate the nut (H-LNUT-M8-NYL) counterclockwise and then, rotate the handle (H-PC1-CLMP-HDL) counterclockwise until the PLV-8-32 can be removed.

## **Troubleshooting**

| What it looks like | What happened                           | How to fix it                 |
|--------------------|---|-------------------------------|
| Adapter is loose   | Adapter screw and/or knob have loosened | Tighten all screws and knobs. |

#### Maintenance

It is recommended that once every three (3) months, all locks, bolts and screws are visually inspected for signs of product wear and tear or damage. Check the product for loses in the mount or orientation, and ensure the adapter is secure.

### Warranty

During the term of a 5-year warranty: Within the first twelve (12) months from the date of shipment, Amico Accessories will repair or replace any part which is proven to be defective at no cost. After the twelve (12) month period, Amico Accessories will send the parts to the customer free of charge, however shipping and installation will be borne by the customer.

The warranty is valid only when the product has been properly installed according to Amico Accessories specifications, used in a normal manner, and serviced according to factory recommendations. It does not cover failures due to damage which occur in shipments or failures which resulted from accidents, misuse, abuse, neglect, mishandling, alteration, misapplication or damage that may be attributable to Force Majeure.

#### AMICO ACCESSORIES DOES NOT HONOR VERBAL STATEMENTS CONCERNING THE WARRANTY.

The distributor and/or dealer are not sanctioned to create verbal warranties about the product described in this agreement. Any statements will not be honored or be made part of the agreement of sale. This document is the final complete and exclusive terms of the agreement.

#### THIS WARRANTY IS INCLUSIVE AND REPLACES ALL OTHER WARRANTIES.

Amico Accessories shall not, under any circumstances be liable for incidental or consequential damages including, but not limited to, profit, loss of sales or injuries to person(s) or property.

Correction of non-compliance as noted above will result in completion of all liabilities of Amico Accessories whether based on agreement, neglect or changed materials, designs or specifications without notice.

All claims for warranty must first be approved by Amico Accessories Customer Service Department: (info@amico-accessories.com or 1-877-264-2697). A valid Return Goods Authorization number must be obtained from Amico Accessories prior to commencement of any warranty claim.