


Installation Instructions

Regal Series Flatwall

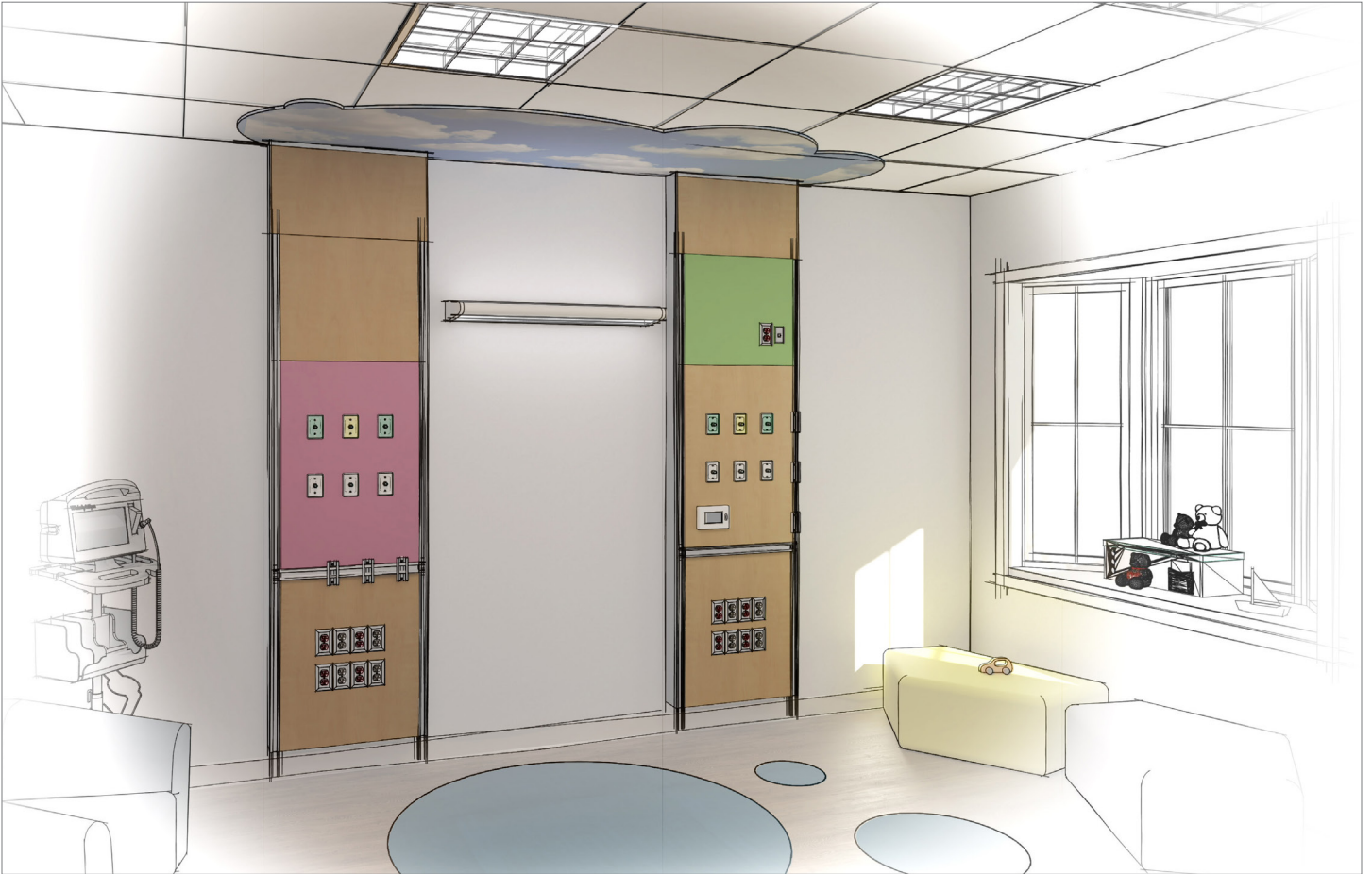

Surface Mounted

Amico

User Responsibility


The installation instructions provided are for the Amico standard products. Slight modifications to the installations may be required. The product shipped to the job site may not be exactly as shown on the installation drawings. See additional sheets for project specifics where required. Consult with project manager regarding custom elements on your project.


IMPORTANT: PLEASE READ CAREFULLY PRIOR TO START


- Review and understand all instructions before commencing work.
- Flatwall installation should conform to the local building codes.

All on-site preparations must be done prior to the delivery of the panel wall units. Please contact Amico to confirm and review all information listed below. Failing to do so may result in discrepancies/misalignments during installation.


Please Note These Keywords and Symbols:

 **WARNING:** Steps where extra care should be taken to prevent injuries and damage.

 **NOTE:** Steps that point out helpful information.

 **IMPORTANT:** Be sure to observe this instruction.

Hardware Provided by Amico

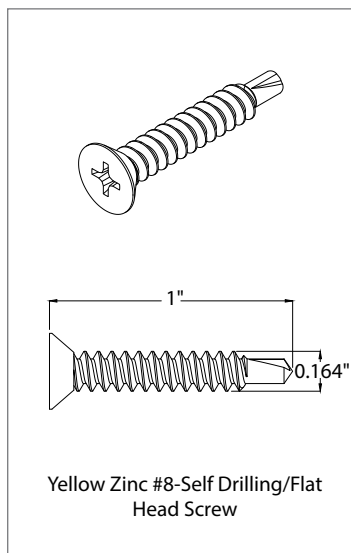

Hardware 1

Preparation Before Installation


1. Review a copy of the final approved shop drawing(s) and/or submittal package. These documents will provide you with technical information specific to your installation, such as:

- Equipment types and quantities
- Room numbers and locations
- Shop drawings of each unit type
- Wiring diagrams and rough-in locations
- Blocking locations


! IMPORTANT: Before installation of units, make sure the blocking required for mounting the headwall is installed at locations indicated on shop drawings.

2. Locate the Flatwall units from the shipping boxes that you wish to install. The contents of each shipping box are listed on the shipping document(s). Please see pictures below for reference:

- Type A Design – Top access panels
- Type B Design – Full length panels

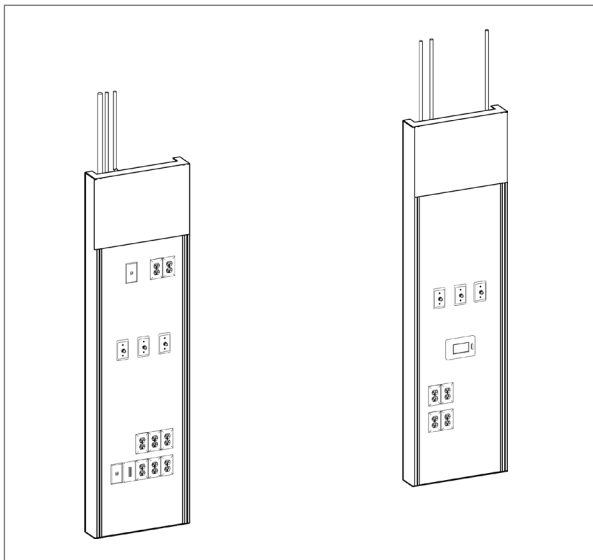

Type A Design
Units with Top Access Panels

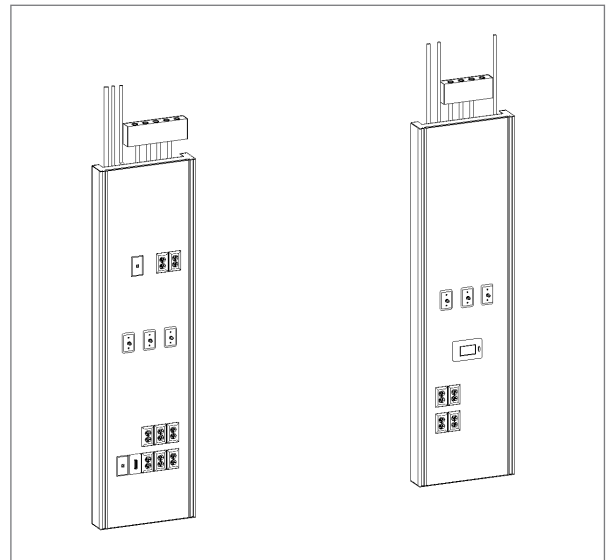

Type A Design
Units with Full Length Panels

Installation

Please choose Step 1 of installation based on the following methods:

Step 1A – Installation with 2 Sets of Hanging Brackets

Step 1B – Installation with Ceiling Grid or Solid Ceiling in Place

Step 1A – Installation with 2 Sets of Hanging Brackets

1. Mark two horizontal lines to locate drilling for mounting of hanging bracket on the wall as follows (**Figure 1**):
 - a. **Under Ceiling Line:** 7" below the ceiling line for the top set of mounting brackets.
 - b. **Above Floor Line:** Refer to the height indicated on shop drawings.

(i) NOTE: These lines will be used to locate drilling for mounting of hanging brackets on the wall. Please refer to the heights indicated on shop drawings.
2. Set up hanging brackets as indicated in technical drawing provided with the units. Brackets are located at the top of each section for ease of identification.

(i) NOTE: Hanging brackets can be found taped to shipped frames.
3. Mount the brackets on the wall using hardware compliant to local building codes, as seen in **Figure 2**.

(i) NOTE: Hardware used to mount bracket to the wall is supplied by others.

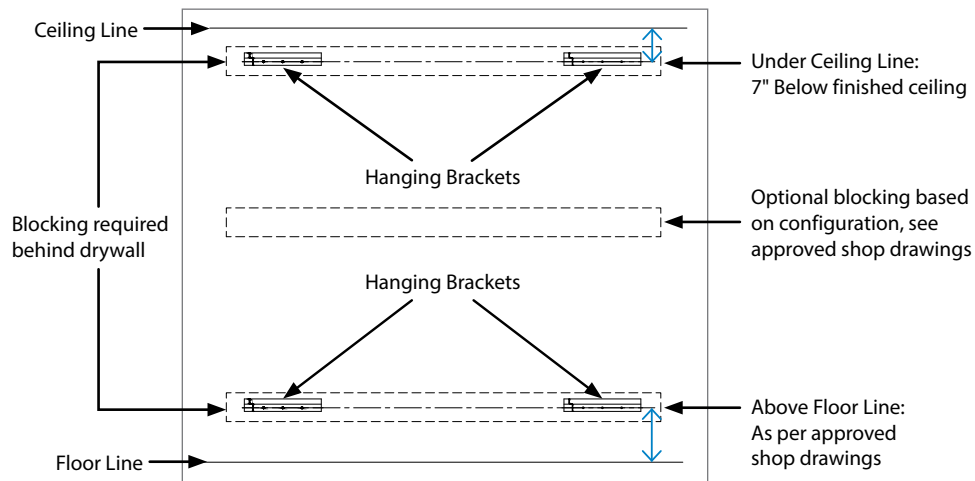


Figure 1

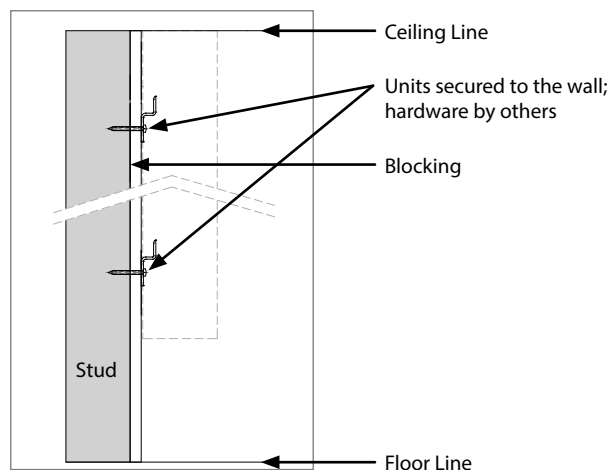


Figure 2: Setup

Installation

4. Remove panels (using tool provided).

! IMPORTANT: See **Page 7** for panel removal and handling instructions.

5. Lift the unit and hang it over the brackets which are already installed on the wall.

i NOTE: Make sure the assembly unit is fully engaged into the hanging brackets at all locations (**Figure 3**).

6. Connect the hanging bracket to the headwall support using the provided self-drilling screws (**Hardware 1**). (**Figure 4**).

i NOTE: The positive mount will be at the top hanging bracket location (**Figure 3**).

7. Make all electrical and piping connections.

8. Attach all removed items:

- Panels (must clip in properly where applicable)
- Devices, extension rings and cover plates
- Rails (must use hardware provided to reinstall)

! IMPORTANT: Do not overtighten the medgas cover plates.

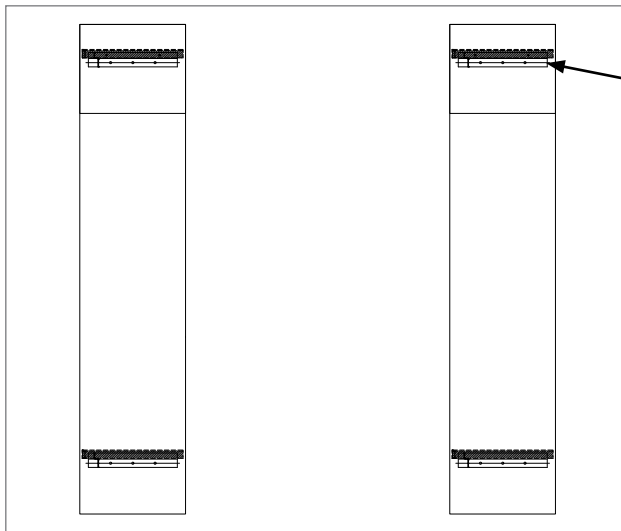


Figure 3: Setup

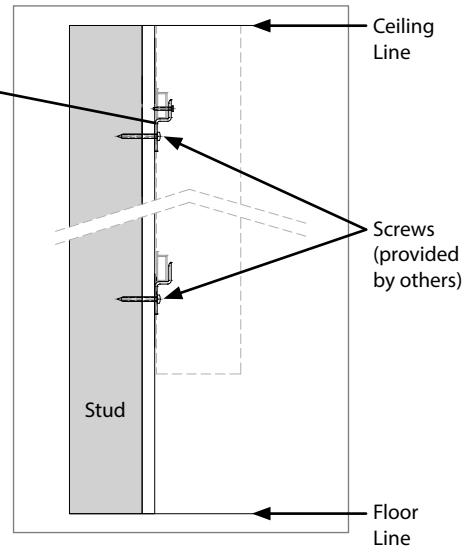


Figure 4: Setup

Installation

Step 1B – Installation with Ceiling Grid or Solid Ceiling in Place

! IMPORTANT: Ceiling tiles must be removed during installation to allow for electrical and medgas piping to pass through.

1. Remove panels (using tool provided).

! IMPORTANT: See **Page 7** for panel removal and handling instructions.

2. Attach the hanging brackets to the headwall support bars designated for hanging brackets.

! IMPORTANT: Do not overtighten the screws – these screws will be tightened more once the unit is attached to the wall.

3. Set the unit in the final location on the wall.

4. Secure the unit to the wall through the hanging brackets on the top.

5. Use hardware compliant to local building codes (provided by others) to mount brackets to the wall, as per **Figure 5**.


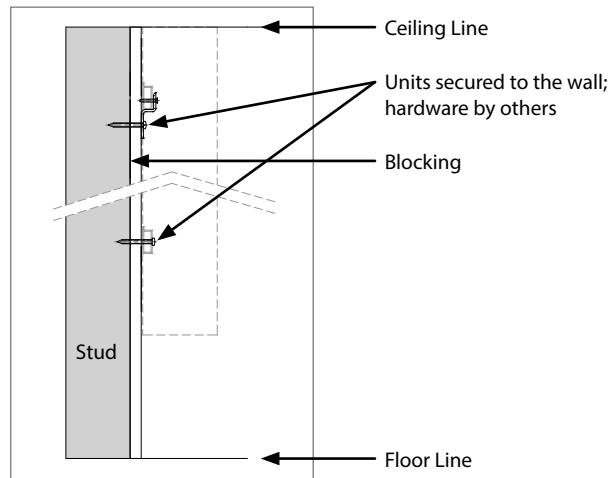


Figure 5: Setup

Panel Removal/Handling

Access Panels

1. Use the provided suction cup to remove the panels. Be careful not to drop panel.

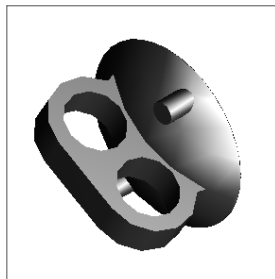

Part#: W-X-LIFTER-0475

Panels with Vertical Rails and Services

1. Remove vertical rails, where applicable:
 - Remove Bottom Plate
 - Remove Insert Strip
 - Remove Screws
2. Remove device cover plate and loosen electrical devices.
3. Remove panels (using the provided suction cup).

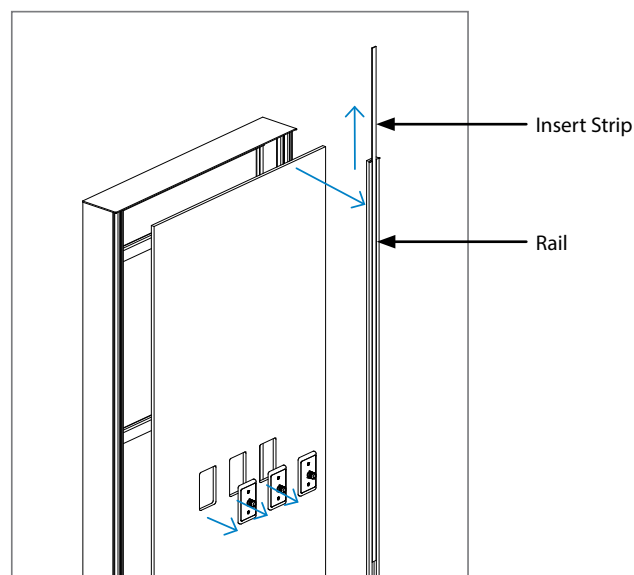


Figure 6

WARNINGS:

- Stack panels face to face to prevent damage.
- Keep panels as flat as possible when handling or storing.
- Do not drag panels on other panels in order to prevent scatching or scuffing.
- It is desirable to keep the panels in a controlled environment where the HVAC system is operating.

www.amico.com

Amico Corporation | 85 Fulton Way, Richmond Hill, ON L4B 2N4, Canada
600 Prime Place, Hauppauge, NY 11788, USA
Toll Free Tel: 1.877.462.6426 | Tel: 905.764.0800 | Fax: 905.764.0862
Email: info@amico.com | www.amico.com



US LISTED

