

# Mobile Aneroid

## Quick Installation and Maintenance Instruction



### Medical Facility Responsibility

Preventive maintenance checks must be performed regularly to maintain the quality and performance of this product. Any parts that may be broken, missing, worn, distorted, or contaminated in any way should not be used and all affected parts should be replaced immediately. Should the necessity of any repair be suspected; please contact your local distributor.

### Installation Tools



### Specifications

#### Physical Dimensions:

(W x D x H): 18" x 18" x 34" (45.7 cm x 45.7 cm x 86.4 cm)

Weight: 13 lbs (5.9 kg)

#### Packaging Dimensions:

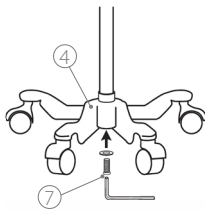
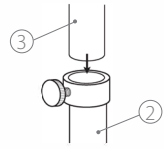

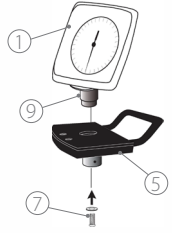
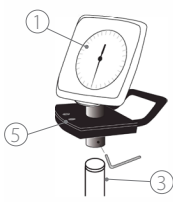
(W x D x H): 16" x 16" x 10" (40.6 cm x 40.6 cm x 25.4 cm)

Weight: 14 lbs (6.4 kg)

### Installation Reference

1. Clock Aneroid
2. Outer Pole
3. Stainless Steel Inner Pole
4. Base
5. Platform
6. Metal Cuff Storage Basket
7. 6 mm Hex Screw & Washer
8. 2x3 mm Hex Screws
9. Clock Aneroid Bracket

### Installation Instructions (AM-RX-LF2118)

1	 <p>With base ④ on its side connect upright outer pole to base using the long 6 mm hex screw and washer ⑦. Tighten with the large hex key provided. Tighten hardware as securely as possible. (To prevent loosening, periodically check that hardware remains securely tightened.) Reposition base upright.</p>	2	 <p>Slide stainless steel inner pole ③ inside outer pole ② with open end up. Make sure shock adsorbing spring (not shown) is inside the outer pole ② and tightened.</p>
3	 <p>Attach metal cuff storage basket ⑥ to platform ⑤ using 2x3 mm hex screws ⑧. Tighten with small hex key.</p>	4	 <p>Attach clock aneroid bracket ⑨ (with attached clock aneroid ①) to platform ⑤ using 6 mm short hex screw and washer ⑦. Tighten with large hex key.</p>
5	 <p>Place assembly consisting of clock aneroid ①, metal cuff storage basket (not shown) and platform ⑤ on top of stainless steel inner pole ③, making sure the steel inner pole goes completely inside collar. Use tightening device and small hex key as shown.</p>	6	<p>Connect coiled tubing (not shown) to gauge inlet (not shown). <b>Note:</b> The coiled tubing may be difficult to connect; moisten with soapy water if necessary.</p>

# Mobile Aneroid

## Quick Installation and Maintenance Instruction

---



### Troubleshooting

What it looks like	What happened	Solution
Stainless steel inner pole is sliding down.	Adjusting knob is loose.	Tighten adjusting knob by turning clockwise.
Pole is wobbling.	Long 6 mm hex screw connecting base to outer pole is loose.	Tighten long 6 mm hex screw.

### Maintenance

It is recommended that once every three (3) months, the product be checked for looseness in the mount or orientation, and ensure the adapter is secure.

### Warranty

During the term of the warranty (3 years), Amico Diagnostic Incorporated will either repair or replace any components determined by Amico Diagnostic Incorporated to be defective at no cost to the Customer. It shall be the Customer's responsibility to return the Aneroid to Amico Diagnostic Incorporated. Shipping costs after the first twelve (12) months shall be borne by the Customer.

This warranty is valid only when the product has been properly installed according to Amico Diagnostic Incorporated's specifications, used in a normal manner and serviced according to factory recommendations. It does not cover failures resulting from accidents, misuse, abuse, neglect, alteration, misapplication, improper maintenance, shipping or damages that may be attributable to acts of God. The term of this warranty begins from date of purchase from Amico Diagnostic Incorporated.

### Calibration Warranty

During the term of calibration warranty (lifetime), Amico Diagnostic Incorporated will recalibrate the aneroid at no charge should the aneroid deviate from the +/-3mmHg accuracy specification. It shall be the Customer's responsibility to return the Aneroid to Amico Diagnostic Incorporated. Shipping costs after the first twelve (12) months shall be borne by the Customer.

Amico diagnostic incorporated does not honor verbal statements concerning the warranty.

The distributor and/or dealer are not sanctioned to create verbal warranties about the product. This warranty is inclusive and replaces all other warranties.

Amico diagnostic incorporated shall not be liable for any loss or damages, whether direct, incidental or consequential.

All claims for warranty must first be approved by Amico Diagnostic Incorporated's Service Department (accessories@amico.com or 1.877.264.2697). A valid Return Goods Authorization (RGA) number must be obtained from Amico Diagnostic Incorporated prior to commencement of any service work. Warranty work, which has not been pre-authorized by Amico Diagnostic Incorporated, will not be reimbursed.